

# H. pylori

Rapid immunoassay for the qualitative detection of Anti-Helicobacter pylori antibodies in human whole blood, serum or plasma



- **CLIA Waived** for whole blood
- Sensitivity of 95.9% with 91.6% overall agreement to biopsy histology
- Easy-to-use single reagent procedure
- Results available in 10 minutes
- Requires only 25µl of whole blood or 10µl serum/plasma
- Convenient finger stick sample collection

# Status H.pylori

Refer to Product Instructions for details.

## STEP1

Remove a test device from its pouch and place on a flat surface.

*H.pylori*

## STEP2

Collect sample using the capillary tube provided and add to test device Sample Well.

- 25µL (red line) for whole blood
- 10µL (black line) for serum/plasma

*H.pylori*

## STEP3

Add 2-3 drops of Developer Solution into Sample Well.

*H.pylori*

## STEP4

Read the results at 10 minutes.

0

10Minutes

## INTERPRETATION OF RESULTS

*H.pylori*

C  
T

C  
**OR**<sub>T</sub>

Two Lines  
Positive  
(+)

**Any visible t  
line is positive**

C  
T

C Line  
only  
Negative (-  
)

C  
T

C  
**OR**<sub>T</sub>

No C Line  
Invalid

**Repeat with new  
sample and device**