


DPP[®] HIV 1/2 Assay

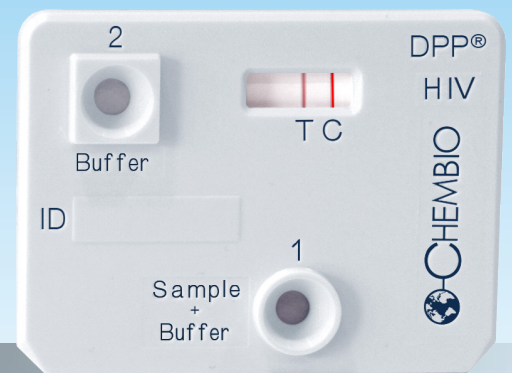

Rapid HIV Test That Works with Oral Fluid & Blood

An FDA approved, rapid point-of-care test that detects antibodies to HIV-1 and HIV-2.

Flexible

- Works with oral fluid, fingerstick blood, venous whole blood, serum and plasma

Convenient

- Low sample volume: only **10 µl**
- Long Shelf life: 23 months
- Safe, closed system for sample handling with DPP SampleTainer[®]

Accurate

- Patented DPP[®] Technology allows for higher sensitivity and specificity
- Built-in procedural control

Test for HIV in 3 Easy Steps with DPP® HIV 1/2

Please refer to the product insert for details

ORAL FLUID

1 Prepare

Swab outer gums for 30 seconds



2 Run


2 drops Sample + Buffer to Well 1



4 drops running buffer to Well 2



3 Read

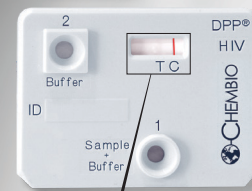

NON-REACTIVE



REACTIVE



INVALID

BLOOD

1 Prepare

Use loop to obtain sample



2 Run

2 drops Sample + Buffer to Well 1



4 drops running buffer to Well 2



Product Performance

Sample Type	Sensitivity	Specificity
Oral Fluid	98.9%	99.9%
Fingerstick	99.8%	100%
Whole Blood	99.9%	99.9%
Plasma	99.9%	99.9%
Serum	99.9%	99.9%

Ordering Information

Product	Catalog Number
DPP® HIV 1/2 Assay (20 tests) FDA Approved, CLIA Waived	65-9500-0
DPP® HIV 1/2 HIV Rapid Test Control Pack	60-9552-0

For More Information

Visit: www.chembio.com
Call: 1.844.CHEMBIO
Email: info@chembio.com