

URISCAN

Urine Test Strips

▪ **CLINICALLY PROVEN QUALITY**

▪ **TEST FOR LESS**

Be wise with your urinalysis \$\$\$\$.

Contact your UriScan dealer today.

ITEM NO.	URISCAN URINALYSIS TEST STRIPS	SIMILAR PRODUCT X-REFERENCE		PRICE
		SIEMENS	ROCHE DIAGNOSTICS	
U15	1 KETONE STRIP (Ketones)	Ketostix	Chemstrip K	
U19	1 GLUCOSE STRIP (Glucose)	Diastix	No Equivalent	
U20	2 GP STRIP (Glucose, Protein)	Uristix	Chemstrip 2GP	
U25	6L STRIP (Blood, Protein, Nitrite, Glucose, Leucocytes, pH)	Uristix 4	Chemstrip 4OB	
U39	10 SGL STRIP (Blood, Bilirubin, Urobilinogen, Ketones, Protein, Nitrite, Glucose, pH, Specific Gravity, Leucocytes)	Multistix 10SG	Chemstrip 10SG	

• Strips can be read visually or by UriScan Optima urine analyzer. For visual reading, dip strip in urine sample for less than 1 second. Blot off excess urine by tapping strip on its side. Read all test parameters other than Leucocytes at 60 seconds. Read Leucocytes at 90 seconds.

• Laminated color charts are available free of charge upon request.

• Urinalysis report forms are available online at www.biosyslabs.com.

Manufactured For:




Los Angeles, California

1.800.365.6146

www.biosyslabs.com

Distributed By:

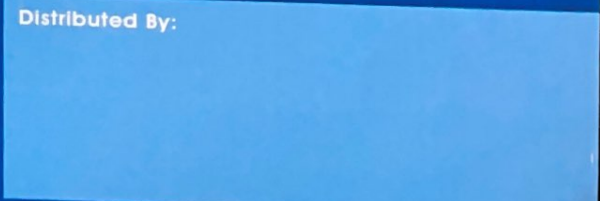

URISCAN OPTIMA

Urine Analyzer



PRODUCT FEATURES

- 36 tests per hour routine mode capacity
- 300 tests per hour quick mode capacity (Results within 6 seconds!)
- TFT LCD display with touch screen
- CLIA Waived
- Sleek, compact design
- For use with UriScan 10SGL, 10ACR, 6L 2ACR and other UriScan urine test strips

PRODUCT DETAILS


Memory capability	Up to 2,000 tests
Result time	100 seconds in routine mode, 6 seconds in quick mode
Test capacity	36 tests/hour in routine mode 300 tests/hour in quick mode (after color development time)
CLIA waived	Yes
Type of strips used	Multiple urine strips, including 10SGL, 10ACR, 2ACR and 6L
One-touch strip selection	Calibration results stored for easy one-touch switching between different test strip configurations
LCD display and touch screen	Yes
Reads color and records clarity of the urine	Yes
Ratios	Calculates albumin-to-creatinine ratio
Bar code reader	Optional
Results downloadable to computer	Yes
Built-in thermal printer	Yes
Dimensions	8" (W) X 9.25" (D) X 3" (H), Weight 2 lbs.

SPECIAL OFFERINGS:

Manufactured For:



Los Angeles, California
1.800.365.6146
www.biosyslabs.com

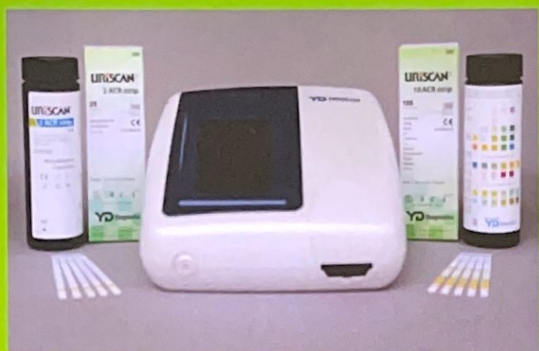

URISCAN MICROALBUMIN/ CREATININE Urine Test Strips

FEATURES

- FDA 510(k) Approved and CLIA-Waived
- Two Configurations of Urine Strips: UriScan 2ACR and UriScan 10ACR
- UriScan 10ACR provides the best of 10SG Plus Microalbumin (MA) and Creatinine (C) test pads
- Calibration results stored for one-touch switching between test strip configurations
- Rapid 90 second test
- Calculates Microalbumin to Creatinine Ratio
- Read with upgraded UriScan Optima Color Touch-Screen Urine Analyzer
- EMR Compatible
- 1 roll of fitted thermal paper included (Reorder #M-TR)
- Special introductory strip and analyzer packages available upon request

ITEM NO.	DESCRIPTION	PRICE
U64	URISCAN 2ACR (25 tests/btl) <ul style="list-style-type: none"> • Tests for: Microalbumin and Creatinine • Calculates Microalbumin to Creatinine Ratio • CPT Codes 82044 and 82570* 	
U62	URISCAN 10ACR (100 tests/btl) <ul style="list-style-type: none"> • Tests for: Leucocytes, Nitrite, Blood, pH, Specific Gravity, Glucose, Ketones, Protein, Microalbumin, Creatinine • Calculates Microalbumin to Creatinine Ratio • CPT Codes 82044, 82570 and 81003* 	
U65	MICROALBUMIN/CREATININE STARTER KIT <ul style="list-style-type: none"> • 1 box U62 UriScan 10ACR Urine Test Strips • 5 boxes U39 UriScan 10SGL Urine Test Strips • 1 each M7 UriScan Optima Touch-Screen Urine Analyzer 	
M7	URISCAN OPTIMA TOUCH-SCREEN URINE ANALYZER <ul style="list-style-type: none"> • Updated MA/C Version • Reads multiple test parameters including UriScan 10SGL, UriScan 10ACR, UriScan 2ACR, UriScan 6L 	

**All CPT codes are presented for information purposes only and represent no statement, promise or guarantee that the codes are appropriate or that reimbursement will be made.*

Manufactured For:



Los Angeles, California

1.800.365.6146

www.biosyslabs.com

Distributed By: